

Surgical technique

1-Retract anterior and posterior walls. Remove all tissue. Ream size for size.

2-Fix trial cup on curved holder/ impactor. Seat trial cup to check the contact with cup back and periphery.




3-Fix holed flanges and hooked flanges with the forceps



4-Curve the with the curved iron.



5-Check the graduation and adjust the holder pliers to the right diameter.

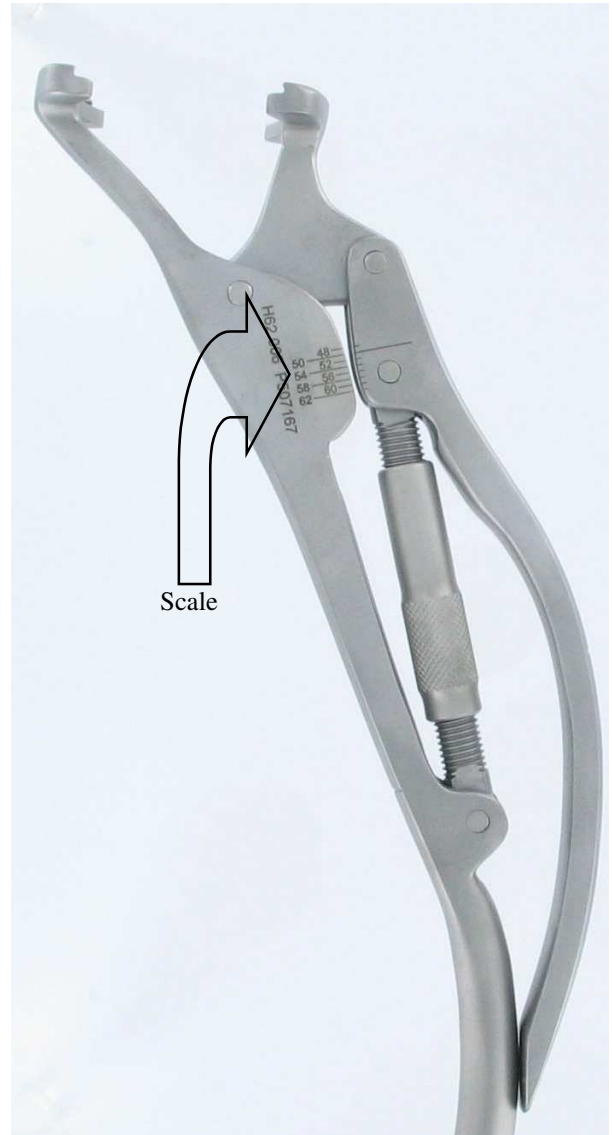

6-Impact the cup in the acetabulum.

Manufactured by :

evolutis

Avenue de la Libération - F-42720 BRIENNON

☎ : +33.4.77.60.79.99 ☎ : +33.4.77.60.79.90

e-mail : evolutis@evolutis42.com

CE
0499

7- Bipolar inlay
 Assemble the femoral head in the inlay with the press.

A product supplied to you by...


SCANOS



Inlay reduction in the cup with impactor.



Instrument set	
DESCRIPTION	REF
Complete CAPITOLE instrument set	H62 9100

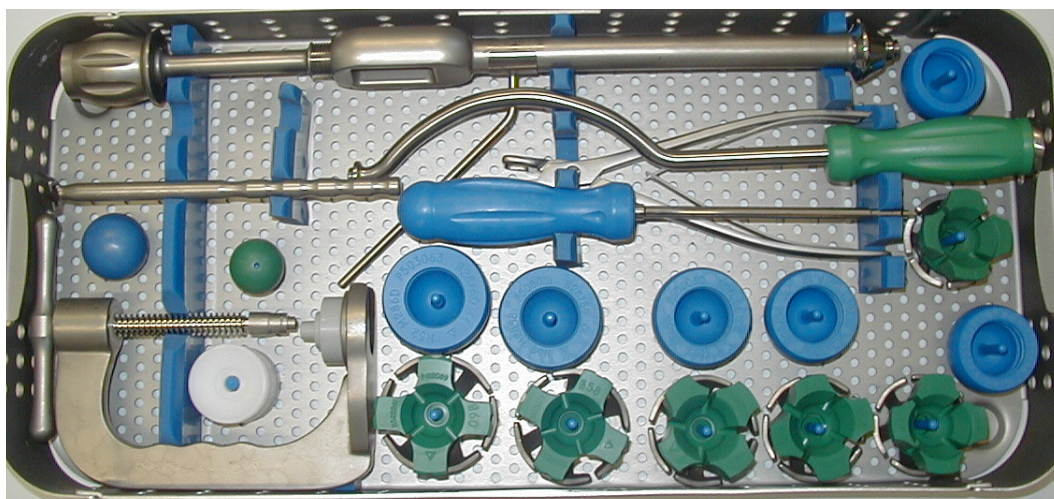

Manufactured by :



evolutis

Avenue de la Libération - F-42720 BRIENNON
 ☎ : +33.4.77.60.79.99 ☎ : +33.4.77.60.79.90
 e-mail : evolutis@evolutis42.com

REF H62 523 - 11/06